
**Testloaded with 120 kg (265 lbs.) static load.  
Maximum elevation 20 cm (7.9 inches).**

**Fits furniture legs:**

1. Square legs up to 6 cm (2.4 inch.)
2. Round legs up to 6 cm (2.4 inch.)
3. Oval legs up to 4x6,5 cm (1.6x2.8 inch.)
4. Rectangular legs up to 4x6,5 cm (1.6x6.5 inch)

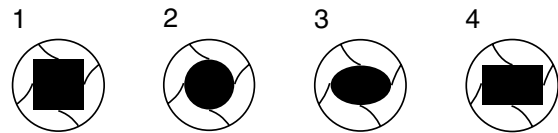
**Consider this when using leg extenders:**

**Important!**

Raising furniture height with leg extenders may involve increased risk for instability. Carefully consider the situation in which the leg extender are to be used, especially with respect to the possibility of tipping. If the patient sits or lies down in an uncontrolled manner, thereby displacing the piece of furniture, the risk of tipping may arise. Note the position of the piece of furniture and assess the need for extra support against one or two walls. Make certain that the leg extender properly fits the piece of furniture on which it is to be used.

Exercise care and caution when using leg extenders. As a caregiver and the person testing the equipment, the patient's safety is always your responsibility. You must be informed of the patient's ability to manage situations involving the use of new assistive devices. Check the leg extenders regularly. Pay particular attention to possible displacement of furniture legs and leg extenders and any damage that may have occurred. Do not use damaged leg extenders. When in doubt, contact the manufacturer or supplier.

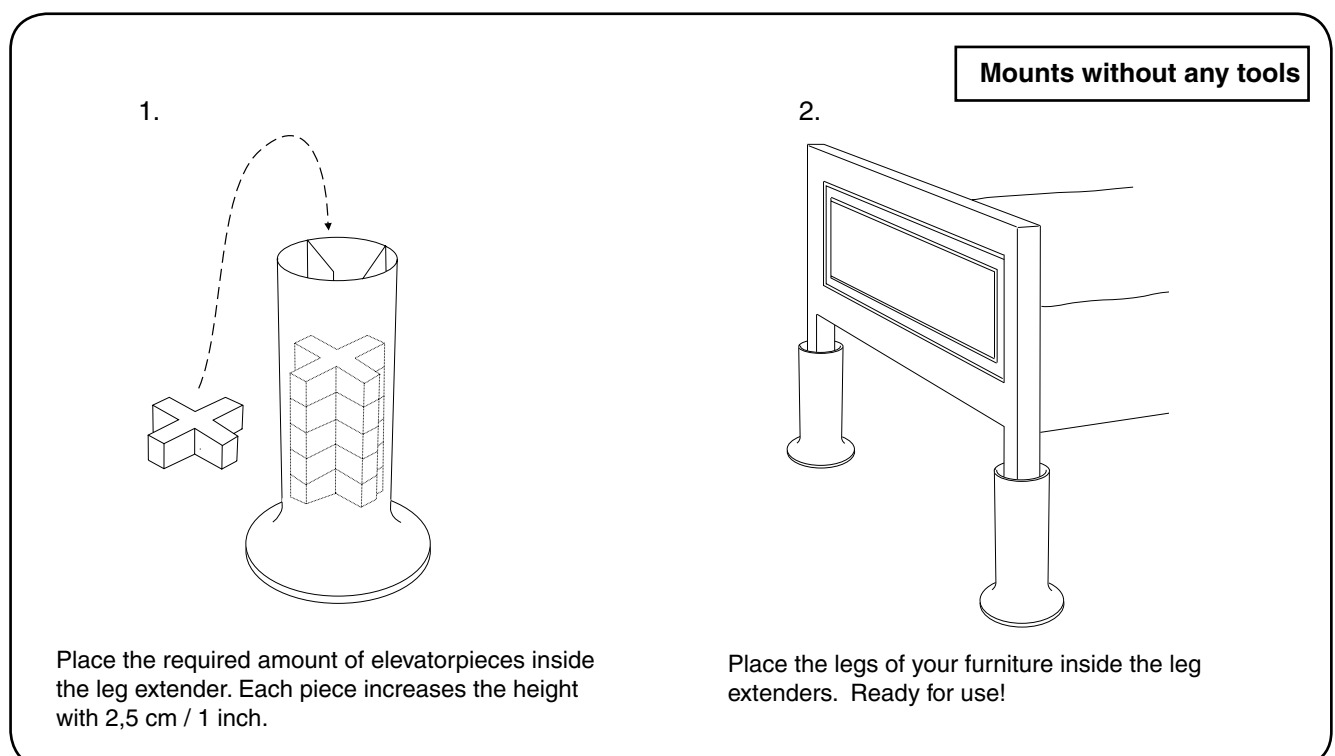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