
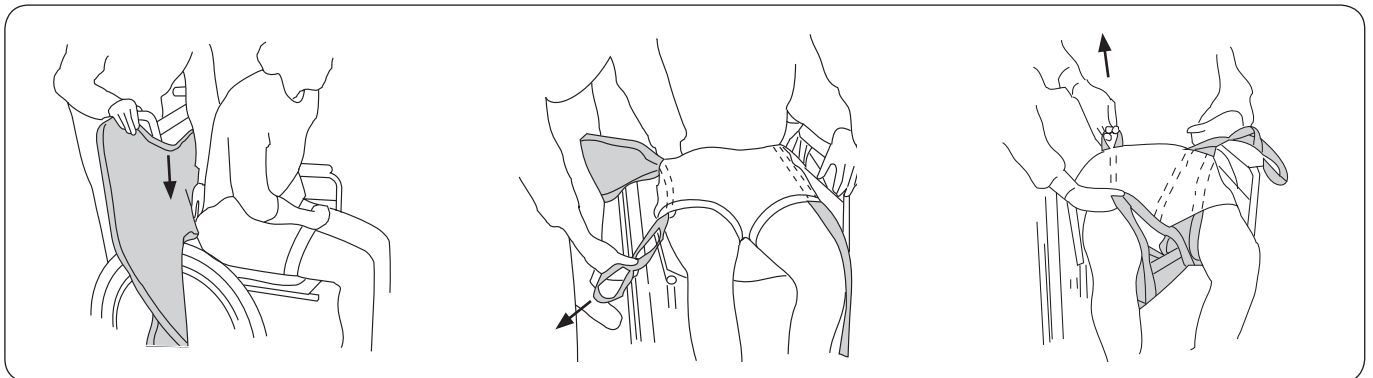


Lifting to the toilet with "the pants trick"

 This instruction is a complement to the instruction guide for UniversalSling, Mod. 00/02 and OriginalSling, Mod. 10/11. Read the instruction guide for both the sling and the lift being used. Instruction guides can be downloaded, free of charge, at www.liko.com.

This method is recommended when lifting to the toilet in an UniversalSling, Mod 00/02 or an OriginalSling, Mod. 10/11. It makes it possible to remove the underpants and the incontinence pad during the actual lifting operation. You thereby avoid an extra lift to/from the bed, where undressing/dressing otherwise would take place.


Conditions for using this method: a woman wearing a dress/skirt that she is not sitting on. In this description we have chosen to show illustrations without the dress/skirt for the sake of clarity.



Lean the patient/let the patient lean forward slightly, so that the underpants can be pulled down towards the seat of the chair. Place the sling inside the underpants and the incontinence pad, if one is worn. Guide the sling down to the tailbone.

Pull the leg supports forward inside the underpants.

Insert the leg supports under each thigh. Make sure the fabric reaches properly around the leg. Pull the leg supports up inside the underpants.

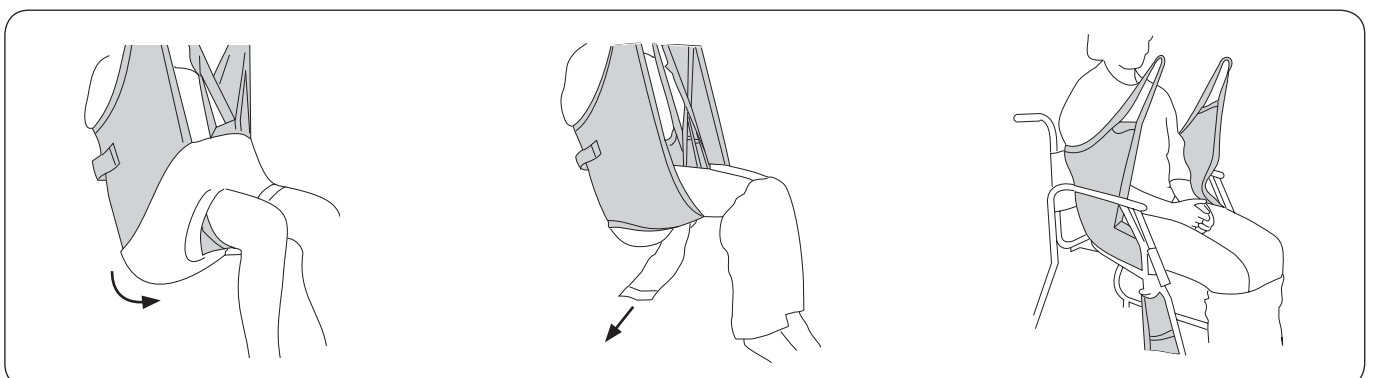


Connect the suspension loops to the sling bar. Lift the patient. Pull the underpants down.

If the patient is wearing an incontinence pad, it can be removed now, or when the patient is sitting on the toilet.

Lower the patient to the toilet or to the hygiene chair. Disconnect the leg support loops from the sling bar. Move the leg supports aside.



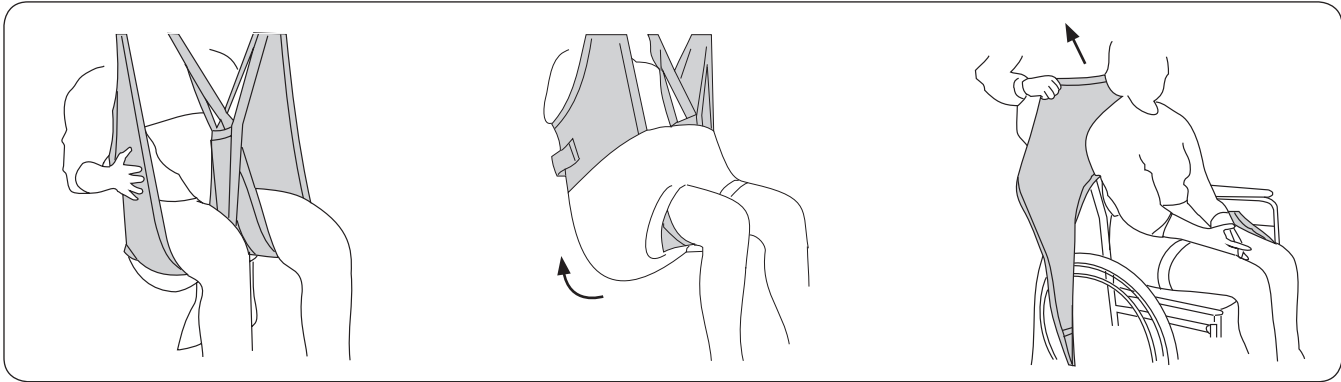

After visiting the toilet:
Insert the leg supports under each thigh. Connect the suspension loops to the sling bar. Lift. If the patient is to wear an incontinence pad, place it inside the crossed leg supports.

Pull up the underpants outside the sling. Lower the patient.

Disconnect the suspension loops from the sling bar. Carefully remove the leg supports. Remove the sling by pulling it up.



A **Hill-Rom** Company

www.liko.com

Manufacturer:

Liko AB
SE-975 92 Luleå
Sweden
info@liko.se